




Correct position of the extraction arm - overview

TABLE MOUNTED ARM

OVERVIEW

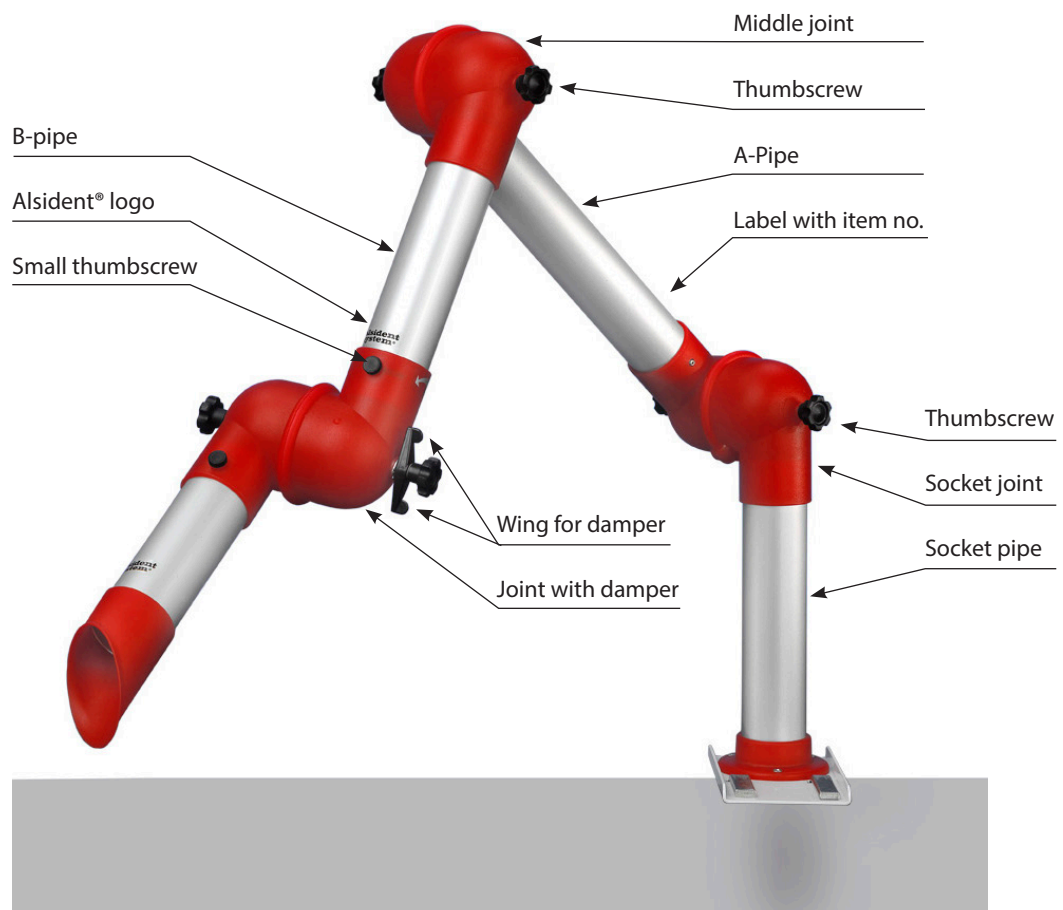



Correct position of the extraction arm


TABLE MOUNTED ARM

CORRECT POSITION OF THE EXTRACTION ARM

We recommend before use to check that the extraction arm is mounted correctly. Incorrect mounting may affect the flexibility of the extraction arm. See illustration on page 3 of the correct mounting.

One of the consequences of an incorrect mounting is that the spring will not come into function and the extraction arm will therefore sink from the intended working position. Through time this will cause the spring to lose its strength irretrievably. To re-establish the correct function of the spring, it will need replacement.

Quick check for correct positioning

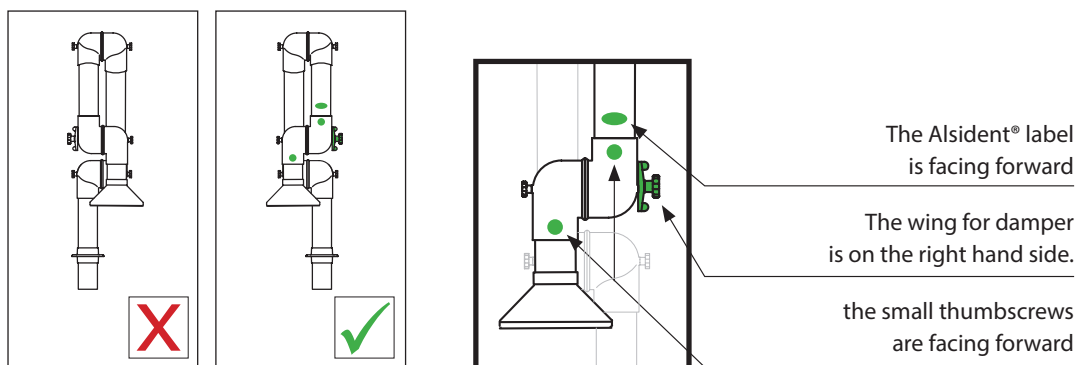

Correcting

This type is easy to correct by turning the two pipes over the socket pipe.

If the extraction arm cannot keep the working position, a strengthening of the spring might help before replacement of the spring.

When the extraction arm operates two work stations, it is important that it is rotated on the socket pipe between the work stations – DO NOT turn the extraction arm up and behind the socket pipe.